



**M**  
 A blue liquid with blue particles is a presumed positive for Methaqualone. Methadone and PCP (Phencyclidine) are false positives in this test.

**N**  
 A slowly developing purple color with the first ampoule, followed by an immediate yellow color with the second ampoule presumptively identifies TALWIN and TALWIN Nx. Talwin is a registered trademark of Winthrop Labs.

**P**  
 A blue color developing with ampoule two, and remaining after breaking ampoule three, presumptively identifies Propoxyphene. If a blue color does not develop after breaking ampoule two, stop test and proceed to Test A for Polytesting. Methadone is a false positive for Test P. Darvon® is a registered trademark of Eli Lilly Co.

**Q**  
 A purple violet color developing after breaking the second ampoule presumptively identifies Ephedrine. Note: The third ampoule is not part of the Q test, but is to be utilized in neutralizing the reagent materials, which are highly caustic.

**R**  
 A slowly developing light violet/purple color in the right ampoule presumptively identifies Valium®. Valium® is a registered trademark of Roche Products, Inc.